
Electric Patient Lift

INSTRUCTIONS FOR USE

Revised: March 5, 2024
Version: A/0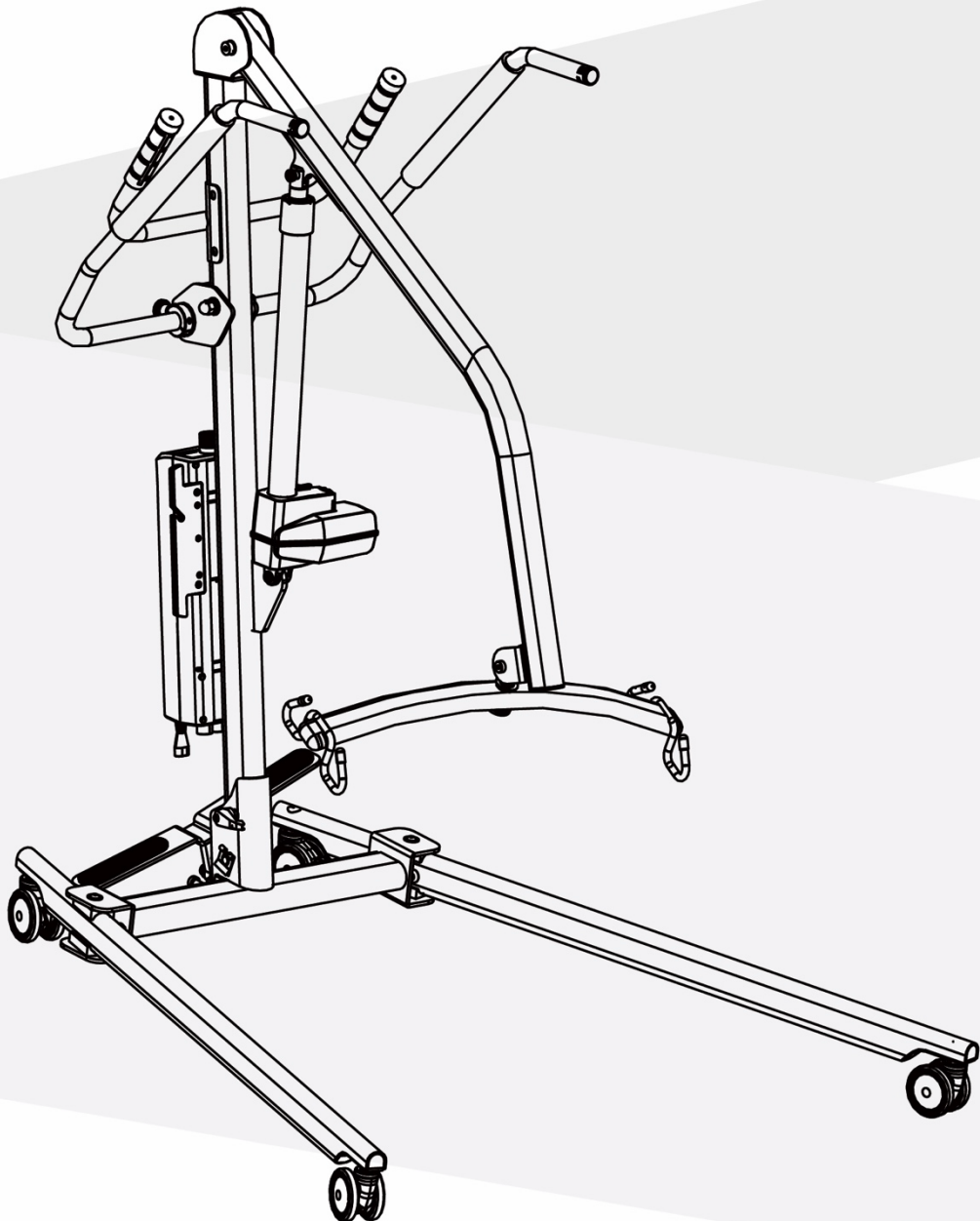

Instruction for Use


Product Name: Electric Patient Lift

Model: PL300, PL301

Intended Use

This device is designed for transferring disabled individuals from one resting surface to another, such as from a wheelchair to a medical bed, and vice versa.

Appearance and Components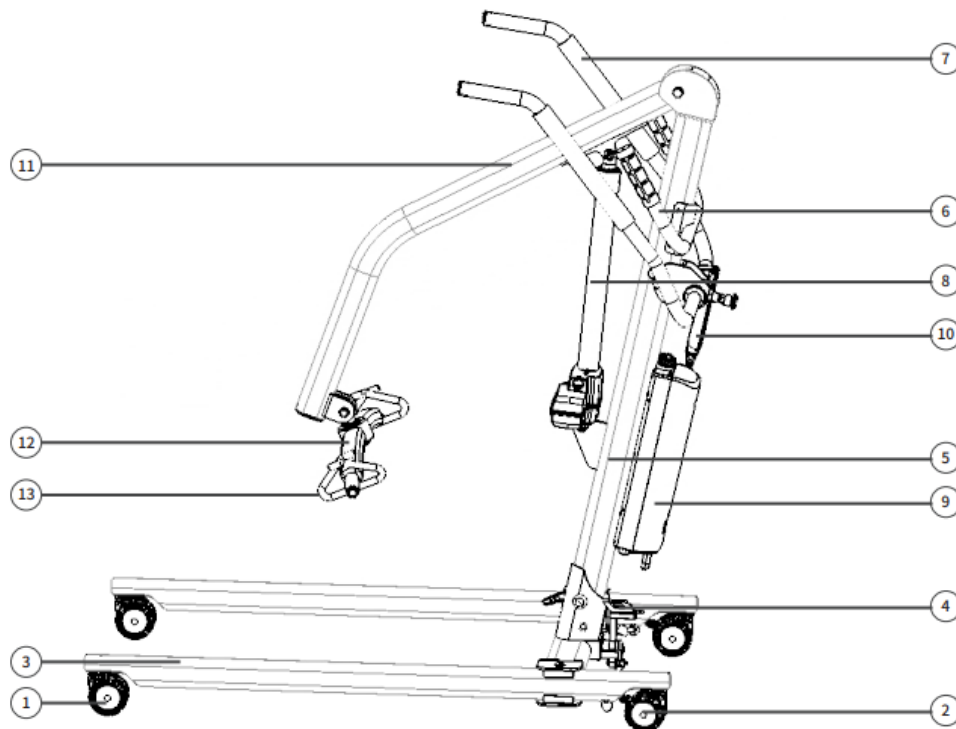

- | | |
|---|---------------|
| ① Front Castors | ⑧ Actuator |
| ② Rear Castors with Brake | ⑨ Control Box |
| ③ Base Legs | ⑩ Handset |
| ④ Leg Opening Pedal | ⑪ Boom |
| ⑤ Mast | ⑫ Sling Bar |
| ⑥ Push handle | ⑬ Sling Hooks |
| ⑦ Rehabilitation Training Armrest
(Optional) | |

Series Product Models

Rehabilitation training armrest is optional.


Model	Product Difference
PL300	With Rehabilitation Training Armrest.
PL301	Without Rehabilitation Training Armrest.

Indications:


This product is intended for use in professional healthcare facilities as well as home healthcare settings. It is not designed for independent use by the patient. Lifting and transferring a patient should always be carried out with the assistance of at least one caregiver.

Assembly Instructions:


1. Connect the upper end of the actuator to the boom using an M8 butt screw^② and secure both of them with a nut^①.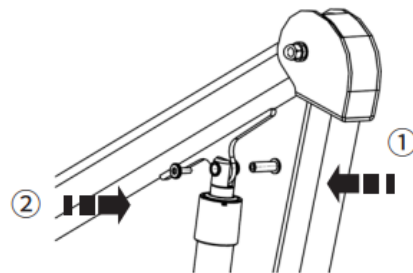


2. Place the mast into the base bracket, then insert the pin and M10 round head square neck bolt^② into the corresponding holes on the base bracket. Secure the internal thread handle^③ and insert the anti-loose block card^①. After installation, ensure there is no noticeable shaking on the mast.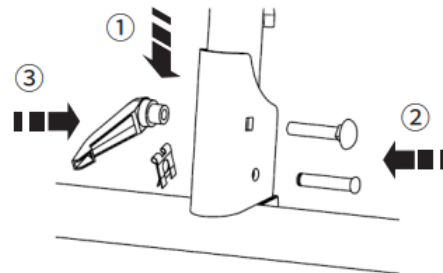

3. Secure the push handle to the mast using M6 screws.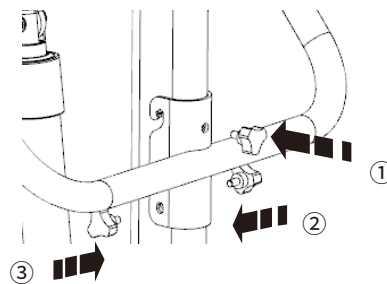

4. Secure the rehabilitation training armrest to the mast using an M12 stud, washers, and cap nuts.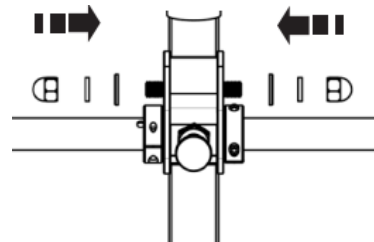

Technical Data:

Size: 120L x 62W x 135H (cm)
Safe Working Load (SWL): 160kg
Max. Base Legs Width: 108cm.
Min. Base Legs Width: 55cm.
Lifting Range: 50-191cm.
Base Legs Height: 13cm.

Weight: 39.5kg
Turning Diameter: 140cm.
Battery: 24V/2.9Ah
Sound Level: $\leq 65\text{dB}$
Actuator: 24V, 8000N push load, 10%
Duty Cycle (2-minute ON, 18-minute OFF)

Usage:**Charging Batteries:**

Lock the castors when charging the battery.

Ensure that the emergency stop button is not pressed.

Treat the mains plug as a disconnecting device from the mains.

Familiarize yourself with the instructions for using the lift and lifting accessories.

Charging Procedure:

1. Connect the charging cable from the control box to the power plug.
2. Connect the power cable to a power outlet (100-240V AC).
3. Once connected, observe the control box lamp:
 - Flashing green lamps indicate charging in progress.
 - Three steady green lamps without flashing indicate a fully charged battery. If not, check the connection between the control box and the charging cable.
4. The battery in the control box fully charges in approximately 6 hours. To protect the battery, do not exceed an 8-hour charging time.
5. Regularly check the battery capacity and promptly charge it if the capacity is low.


Notes:

1. Before the lift is used for the first time, charge it for at least 6 hours.
2. Recharge batteries regularly. We recommend daily charging when the lift is in use.
3. Do not activate the emergency stop button during the charging process.

Handset:

The handset features two buttons on the front, controlling the lifting and lowering movements of the boom. In standby mode, with the working status indicator off, pressing a button activates the lift. Both the working status and battery indicators illuminate. Motion ceases upon button release.

On the back of the handset, there's a triangular groove. To lock or unlock the handset, insert the key into the groove and turn it 90°, aligning it with the mark on the handset.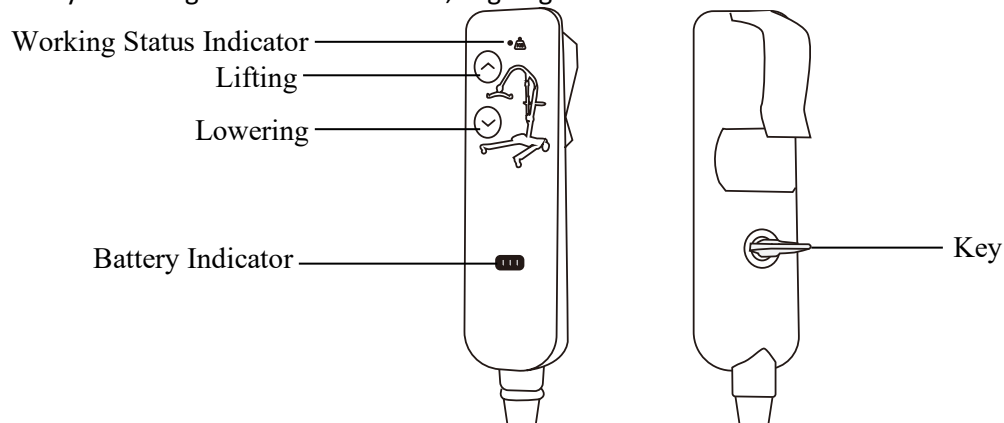

Troubleshooting Guide:

The lift undergoes inspection before delivery. Failures encountered during use may be attributed to the following situations.

Problem	Possible Cause	Solution	Remark
No response when pressing the handset keys.	Actuator plug loose	Tighten the plug	Self-test
	Handset plug loose	Tighten the plug	Self-test
	Battery is dead	Charge in time	Self-test
Unable to push the lift	Castors can't be unlocked	Check the brakes or replace new castors	Self-test/repair
	Castors are stuck	Check the base and avoid obstacles.	Self-test
Battery can't be charged	Power cable failure	Contact your dealer	Repair
	Battery failure	Replace battery	Repair

Cleaning Methods/Maintenance:**Cleaning:**

To maintain cleanliness, use only pH-neutral detergents to wipe the lift. Ensure prompt drying after wiping and check that the castors are free of dirt and hair.

Maintenance of Important Parts:

- The sling, made of flame-retardant material, is washable and deformation-resistant. Follow the washing conditions specified on the sling label for disinfection and cleaning.
- Regularly inspect the fabric, seams, and straps for any signs of damage, wear, or potential failure.
- Check the base legs for easy widening, ensure castors turn flexibly, and confirm brakes are quick and effective.
- Inspect connections for looseness, deformation, rust, or any other issues. Check structural parts for wear, cracking, or deformation.

Storage:

Store the lift in a dry, well-ventilated area to prevent exposure to sunlight, rain, and corrosive chemicals (oil, acid, alkali). Protect the lift from accidental impact and shield the handset from liquid damage.

Precaution and Warning:

Please read and follow the safety precautions listed below to ensure a simple and trouble-free lifting operation.

Read and understand this instruction before using the lift.

WARNING: Important Safety Information

WARNING:

- ***ALWAYS** use the lift indoors.
- ***ALWAYS** familiarize yourself with the operating control and safety features before lifting a patient.
- ***DO NOT** use a sling unless recommended for use with the lift.
- ***ALWAYS** check the sling's suitability for the patient's size and capacity.
- ***NEVER** use a frayed or damaged sling.
- ***DO NOT** load sharp objects to prevent scratching and safety hazards.
- ***DO NOT** overload; the safe working load is 160kg.
- ***ALWAYS** verify the lift's safe working load for the patient's weight.
- ***ALWAYS** carry out lifting operations according to the user manual instructions.
- ***DO NOT** lift a patient with castor brakes unlocked; let the lift find the correct center of gravity.
- ***DO NOT** attempt to move the lift by pushing on the mast, boom, or patient.
- ***ALWAYS** move the lift with the provided push handle.
- ***ALWAYS** lower the patient to the lowest comfortable position before transfers.
- ***DO NOT** push the lift over uneven or rough ground, especially if loaded.
- ***DO NOT** attempt to push/pull a loaded lift over a floor obstruction the castors can't ride over.
- ***DO NOT** park a loaded lift on ANY sloping surface.
- ***DO NOT** use electric lifts in a shower.
- ***DO NOT** charge electric lifts in a bathroom or shower room.
- ***DO NOT** lift a patient unless you are trained and competent to do so.
- ***ALWAYS** cut off the power and press the emergency stop button when not in use.
- ***DO NOT** operate the lift repeatedly for an extended time without interruption after loading; this may damage the actuator.

Operation Conditions:

For optimum performance, operate the lift within the following condition range:

- a) Ambient temperature: 5°C to 40°C
- b) Relative humidity: 20% to 80%
- c) Atmospheric pressure: 80kPa to 106kPa
- d) Charging input: 100-240Vac, 50/60Hz; Discharging output: 24Vdc
- e) No potential electromagnetic interference with other devices

Transportation and Storage:

During transportation or when the lift is not in use for an extended period, push in the emergency stop button.

The environment for transporting and storing the lift should meet the following conditions:

- a) Ambient temperature range: -10°C to +45°C
- b) Relative humidity range: Less than 90%
- c) Ensure no corrosive gases and maintain good ventilation.

Package:

The package consists of one set per master carton.


Warranty:

The lift is guaranteed for one year from the date of purchase under proper use and service.

Expected Lifetime:

The lift has an expected service life of 5 years when handled, serviced, and inspected correctly according to the provided instructions.

Labels, Packing Logo Design

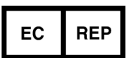Symbol	Introductions	Symbol	Introductions
	Serial number		Date of Manufacture
	Manufacturer		Name and Address of European Union Representative
	Follow instructions for use		Type B applied part
	Class II equipment		Detachable part weighs more than 10kg
IPX4	Ingress Protection Rating		Do not dispose in general waste
	Medical device		CE Symbol
	UK Conformity Assessed		

Manufacturer

Company: Wiseton Industries Limited
Address: No.9 Jiahe Road, Jiashan County, Jiaxing City, Zhejiang Province, P.R. China
Tel: 0086-0573-84991533

European Authorized Representative

SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands

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Azienda: Wiseton Industries Limited
Company: SUNGO Europe B.V.